Think **OVER**  
Advanced dental unit

# Indice

---

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# *Index*

---

- 02 *Over: innovation and beyond*
- 06 *Dental chair*
- 10 *Instrument table*
- 14 *Assistant table*
- 16 *Water unit*
- 18 *Hygiene systems*
- 20 *Foot control*
- 22 *Operating lights*
- 24 *Create your ideal Over*
- 26 *Equipment*
- 27 *Upholstery colours*
- 28 *Dimensions*

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*The Over is Tecnomed Italia's first top-of-the-range dental unit designed and made entirely in Italy. Tecnomed's researchers were committed to using only the latest technology and the very best materials in the development of Over. The result is a superb dental unit with highly professional equipment that stands out from all other units on today's market.*

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#### **WORLD EXCLUSIVE**

- "Servo System Sprido" servo-assisted cord arms
- Handpiece safety cords

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#### **NEWS FROM TECNOMED**

- Polyurethane body finished in disinfectant resistant paint
- Trendelenburg position
- Technical gel padding
- Assistant table with retractable suction tube

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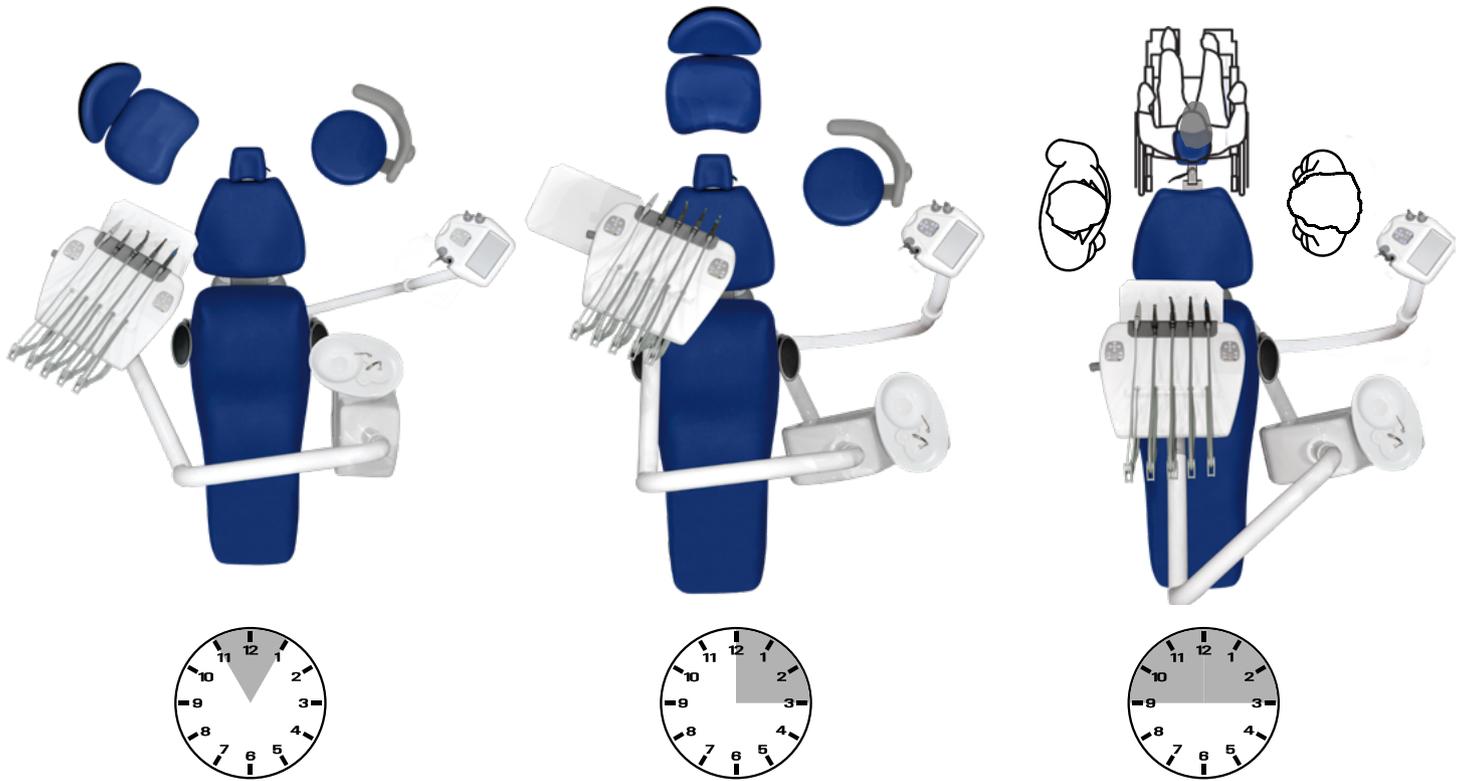
*All the solutions you need from one unit*

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### **CHAIR**

*A die-cast aluminium frame and anatomic upholstery make this chair particularly light and comfortable. A seat height of only 42 cm ensures easy access for the elderly and for children. As an option, you can request a non-toxic technical gel padding that is not prone to memory effect. The backrest and seat move independently and can be controlled by a foot joystick located at the base of the dental unit. The chair is equipped with six anti-crush safety sensors.*

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### **HEADREST**

*The double jointed, fully adjustable headrest can be arranged in any position in just a few quick and easy moves. The ergonomically designed and pivoting headrest moves on a sliding mounting. For the patient, it offers exceptional comfort thanks to anatomic padding that ensures perfect cervical support. For the dentist, it offers high precision and efficiency.*

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### **HEADREST REVERSAL**

*When working with disabled patients, you can pull the headrest off its mounting and fit it the other way around. See the figure on page 04.*

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### **LARGER HEADREST**

*The dental chair can also be fitted with a "moving L" double-jointed headrest instead of the standard model. This headrest is specially designed to ensure maximum visibility inside the oral cavity during operations. It comes complete with a magnetic cushion for patient comfort. The cushion can also be applied to the backrest of the chair when working with children.*

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### **TRENDELENBURG POSITION**

*The chair can be moved automatically into the Trendelenburg position either by pressing a dedicated button on the control panel. Maximum safety for the patient: if a patient feels unwell, the backrest reclines to stimulate the circulatory system.*

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### **ARMREST**

*The chair has a fixed left armrest but can be fitted with an optional movable right armrest to facilitate patient access and descent.*

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### INSTRUMENT TABLE

*The body of the instrument table is in 2 cm thick polyurethane and is designed to hold up to 5 handpieces. The table is supported by a double jointed self-balancing arm. Two control panels with soft-touch keys permit quick and easy selection of available functions even for left handed dentists. The rubber mat under the instruments can be removed and disinfected in an autoclave at 135°C.*

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### INSTRUMENT TABLE CONTROL PANELS

*The instrument table incorporates two control panels, one on either side. Soft-touch keys permit quick and easy selection of available functions. The practical location of the control panels ensures ease of use even for left handed dentists. In addition to the controls provided on previous versions, these latest control panels also feature a patient SOS button that moves the chair into the Trendelenburg position and an operating lamp on/off button. The dental unit's lamp also comes on automatically as soon as the chair is moved to a working position.*

**PANEL ICONS FOR 14 CONTROLS:** glass water, cuspidor water, boiler on, operating lamp on, Trendelenburg position, manual chair and backrest movements, 3 preset positions, position reset, last position.

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#### **HANDPIECE SAFETY CORD**

*A green LED on the cord ring nut lights to warn you when the handpiece needs lubricating, reducing the risk of failures. The smart cord comes as standard equipment. Patent nos.: AN2013A000045, EP14157497.0*

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#### **SPRIDO SERVO SYSTEM**

*As you move the instrument, its cord automatically extends to full length if needed. This technology completely eliminates all pull on your hand while allowing you to exploit the full length of the cord. S.S.S.P.R.I.D.O (Servo System Sprido) servo-assisted cord arms are standard equipment. Patent nos.: AN2013A000046, EP14157517.5*

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### **INSTRUMENT TRAY**

*The aluminium tray is 20 x 30 cm in size. The tray is mounted on a double jointed arm that lets you position it exactly where you need it. It therefore serves as a practical link between dentist and assistant for the exchange of operating instruments. A 30 x 40 cm tray is available as an optional.*

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### **INDUCTION MOTORS**

*Compact, lightweight, silent, perfectly balanced and brushless, micromotors are powerful new maintenance-free three-phase induction motors with INTRAmatic connectors. An efficient winding pattern allows overall length to be kept to a minimum. Advanced LED technology generates high intensity light and guarantees a stable, 25,000 lux illumination of the operating area. The micromotor also incorporates a non-return valve for spray water to eliminate the risk of contamination of the dental unit's water circuit.*

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### **ASSISTANT TABLE**

*The assistant table is mounted on an articulated arm. A special arm mounting system extends the range of movement and permits unrestricted access to the operating area even from the 10 o'clock position. The push-button functions on the assistant's table duplicate those on the dentist's instrument table, allowing assistants to select whatever function is needed. The table has four instrument slots and a handy 18x10 cm aluminium tray. In addition to the syringe and suction cannulae, it is designed to accommodate the curing light.*

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### **RETRACTABLE SUCTION TUBE**

*The main suction tube often gets in your way, and its corrugated surface facilitates the build-up of dirt and bacteria. The Over dental unit's suction tube is routed inside the arm of the assistant table. The filter is hidden from view under the assistant table, but easily accessible and quick to clean. Just pull out and clean the filter! The dental unit comes with two spare filters.*

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### CERAMIC CUSPIDOR

*The cuspidor swings through 105° to make it easier for the patient to rinse. A smooth ceramic surface ensures perfect hygiene. The fountain and the cuspidor are removable and autoclavable for maximum hygiene. The bowl water can be operated either individually or automatically when the cup water is switched on.*

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### WATER UNIT

*The water unit is stylishly designed and made from 2 cm thick polyurethane for excellent resistance to disinfectants. The system inside is mounted on pull-out rails and therefore easy to inspect. The unit also rotates through 150° to make your assistant's work easier. On request, the water unit can be equipped with a Metasys Wek Light water decontaminator.*

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### PURIFIED WATER SYSTEM

*The Purified Water system, fitted as standard equipment, lets you effortlessly select the source of your spray water: mains supply or a pressurised tank containing distilled or disinfected water. The system also prevents contaminated water spray from entering the dental unit, safeguarding your patients' health.*

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### WEK LIGHT

*Leaking instruments and an unpleasant odour from the water are often the first signs that something is wrong with your dental unit's water tubing. More often than not the problem is caused by the formation of biofilms on the internal walls of the tubes. If the water inside the tubes becomes contaminated, it provides a home for bacteria and presents a real risk to the health of your patients and to the effectiveness of the treatments you offer. Biofilm is not only a source of infection but also damages your dental unit and your instruments. Proper water decontamination is therefore indispensable for the efficiency of your dental unit. The Wek water decontaminator from Metasys provides continuous and effective sterilisation of your dental unit's water tubes and prevents the formation of biofilm and the spread of micro-organisms.*

*Wek Light does not require any special maintenance. A buzzer sounds when the bottle of GREEN&CLEAN WK liquid runs out.*

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### SANIVAC

*Sanivac is a system for perfect cleansing and disinfection of suction tubes in 15 seconds using a specific liquid from a dedicated tank. Tecnomed Italia units are uniquely safe and functional thanks to the great attention paid to eliminating all risk of infection and to facilitating operation and maintenance.*

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### **FOOT CONTROL**

*The main feature of the pedalboard is the Feed-Fec control, which allows the instruments to be operated progressively. Spray actuation can always be on by pressing the central button and activating the progressive lever. It can also be activated, if necessary, by pressing the lower lever only. It also features a button for reverse movement of the micromotor. By moving the lever, with instruments inactive, the operating light can be switched on or off.*

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### **JOYSTICK**

*The joystick on the base of the chair can be used to position the chair manually, to move it to its preset positions and to return it to its "zero" position. The operating lamp comes on automatically when chair movement stops. The "S" button returns the chair to its last position, permitting easy rinsing and return to operating position.*

Lampada operatoria  
*Operating lights*

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## ESA LINE

*The Esa Line consists of two shadowless operating lamps: Esalight 2 and Esacam. The lights can be applied to the dental unit, to the ceiling, to the wall or on a wheeled or fixed cart. The ceiling or wall mounted option can support double head lamp (in this case it is possible to join Esalight2 + Esacam).*

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## ESALIGHT 2

*Its brightness is higher than other similar lights and adapts to any treatment thanks to the adjustable light intensity from 8,000 to 60,000 Lux (up to 120,000 lux in the 2-head version). The sensor, positioned near the handle, lets you turn on, turn off and adjust the intensity of the light without touching the light itself, making it both practical and hygienic.*

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## ESACAM

*Both lights have identical optical and structural data. Esacam has a built-in Full HD video camera with a 30x optical zoom and autofocus. This means that allows you to view details that are difficult to see with the naked eye. Thanks to the HDMI output it is possible to connect different devices depending how you want to utilise the images: view, record, project, share, and stream.*

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### MICROMOTOR CONTROL PANEL

*The control panel automatically recognises which instrument is in use and lets you adjust torque and maximum speed, and even reverse direction. Once the handpiece's gear ratio has been selected, the system automatically calculates its speed and displays it on the LCD display. Scaler functioning mode and vibration intensity can also be selected. The LCD panel can also be operated from a multi-functional foot control.*

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### COMMUNICATION KIT

*The complete Tecnomed Italia communication kit consists of a Tecnocam intraoral camera, a Visiomed monitor or PC all in one touchscreen and an arm with handle, to facilitate handling. Thanks to excellent detail reproduction and perfect image quality, this Kit ensures an optimal view of the working area, simplifying communication with your patients. The whole kit is completely cleanable.*

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### DENTAL STOOLS

*Comfortable and ergonomic, our stools respect your body's anatomy and stand out for their excellent quality-price ratio. The height adjustment levers are easy to reach and let you adjust seat position without having to interrupt treatment. Our range includes various different colours and models for a perfect match with the finish and materials of your dental chair.*

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